# Basistraining Tandem Source

# Was ist Tandem Source?

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CGM summary	
Average reading	133 mg/dL
Time in range	86 %
Time CGM in use	100 %
Standard deviation	37 mg/dL
Coefficient of variation	28 %
GMI	6.5 %

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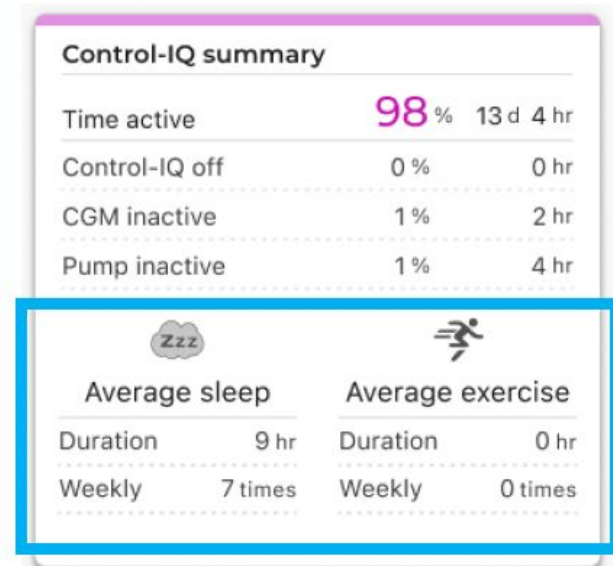
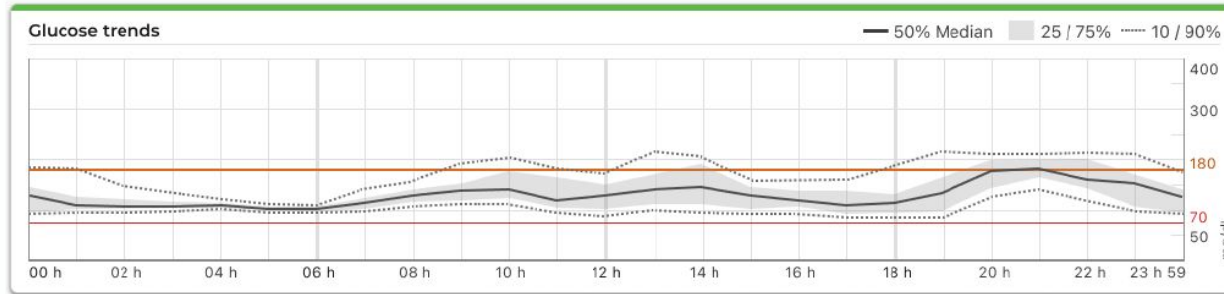
Insulin summary		
Average daily dose		<b>30.29</b> units
Basal	46 %	14.03 units
Bolus	54 %	16.26 units
Average daily boluses		<b>8</b> boluses
Manual	61 %	5 boluses
Control-IQ	39 %	3 boluses
Average daily carbs		<b>96</b> g

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
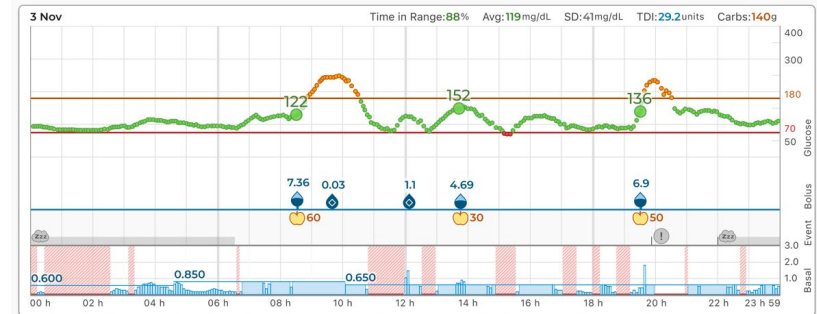
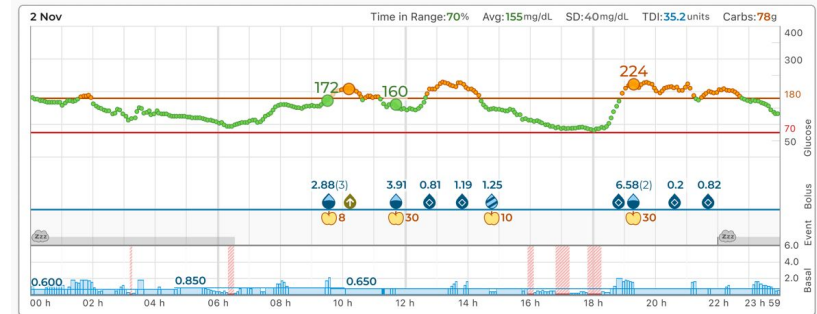
Load activity	
Cartridge change	every 2.9 d
Tubing fill	every 2.9 d
Cannula fill	every 2.5 d

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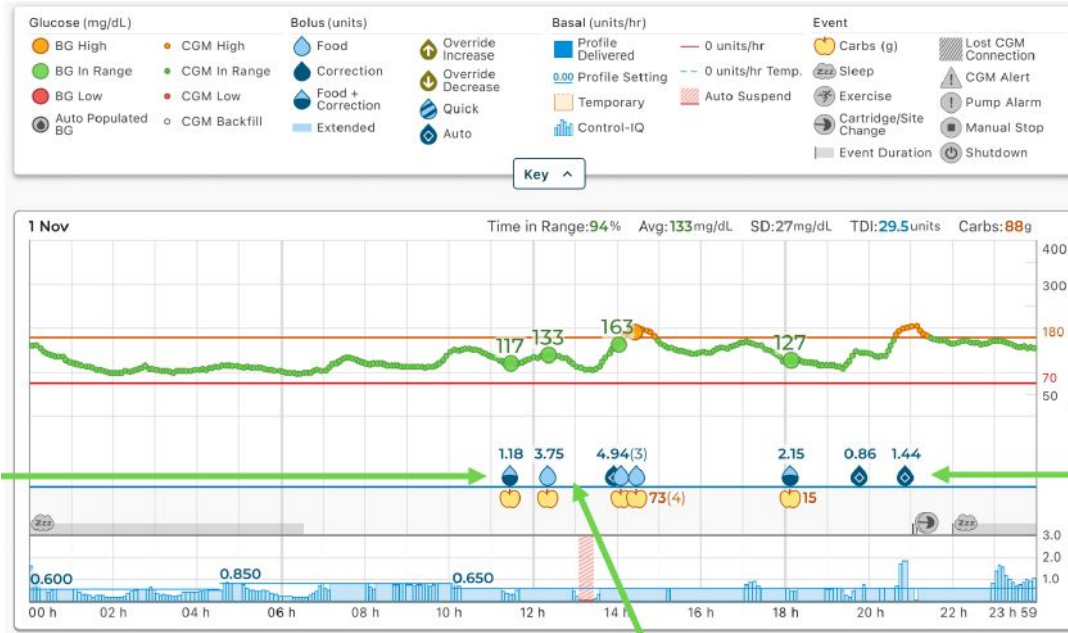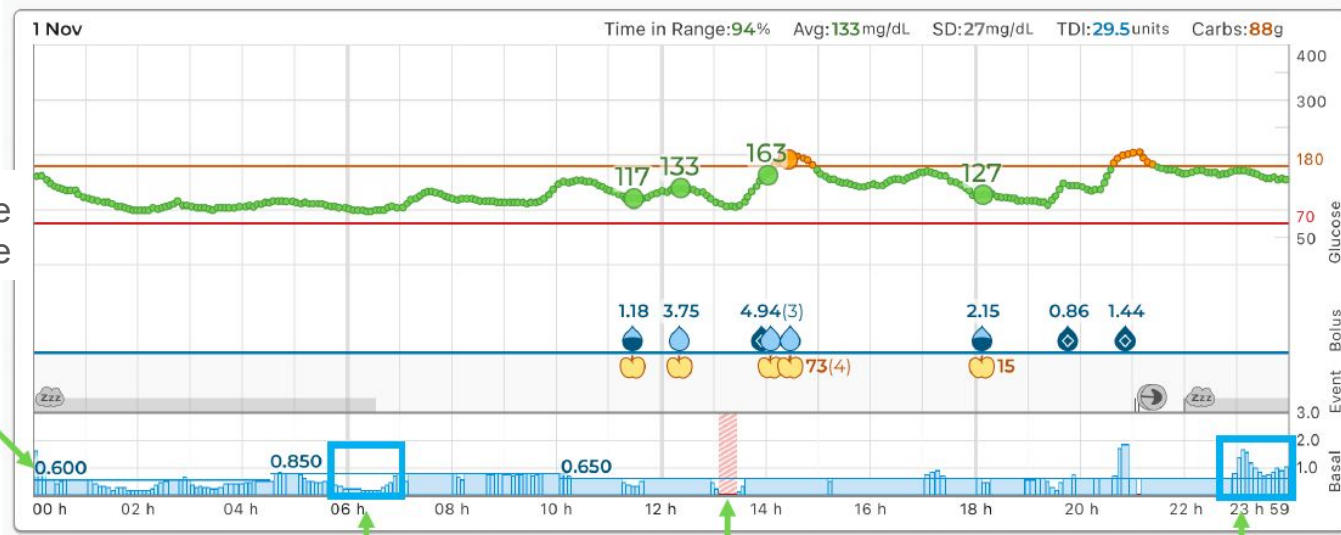
Total Daily Basal: 21.400 units    Insulin Duration: 5 hr    Carbohydrates: On

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Already have an account? [Login](#)

Select the type of account you need

- Personal Use  
I will be using this account for my own personal care
- Parent / Guardian Use  
I will be using this account as a parent, guardian, or caretaker of another person.
- Professional Use  
I will be using this account as a healthcare provider.

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## Upload Pump

Link pump to continue your upload.

Connected Pump: t:slim X2, Control-IQ (v7.6.0.1)  
Pump Serial #: 123456

Where is the pump serial number?

 If this is your pump, please link it to your account to upload and view data. Pumps may only be linked to one account.

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## Share reports with clinic

By sharing you will allow this clinic to view your reports until you choose to stop sharing with them.

Province / Territory  
British Columbia

Search for clinic name or address  
Sta

- Stack Endocrinology - 1234 Maple Street, Vancouver, BC, V6B 1YO,
- Star Endocrinology - 1234 Main Street, Vancouver, BC, V6B 1YO, CA
- Stazz Endocrinology - 1234 University Lane, Vancouver, BC, V6B 1Y

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
The screenshot displays the Tandem Source web interface. On the left is a navigation menu with 'Home', 'Patient List', and 'Upload Pump'. The main content area is titled 'Upload Pump' and shows patient information for 'Tom Tandem', including his date of birth (Mar 20, 1980), last upload time (Oct 31, 2022, 2:45 pm), and the connected pump model (t:slim X2, Control-IQ (v7.6.0.1)). A progress bar indicates that 60% of the data is processed, with the text 'Processing data...' below it. An illustration on the right shows a laptop connected to a pump device. The top right corner of the interface contains a help icon and a user profile icon labeled 'EE'.

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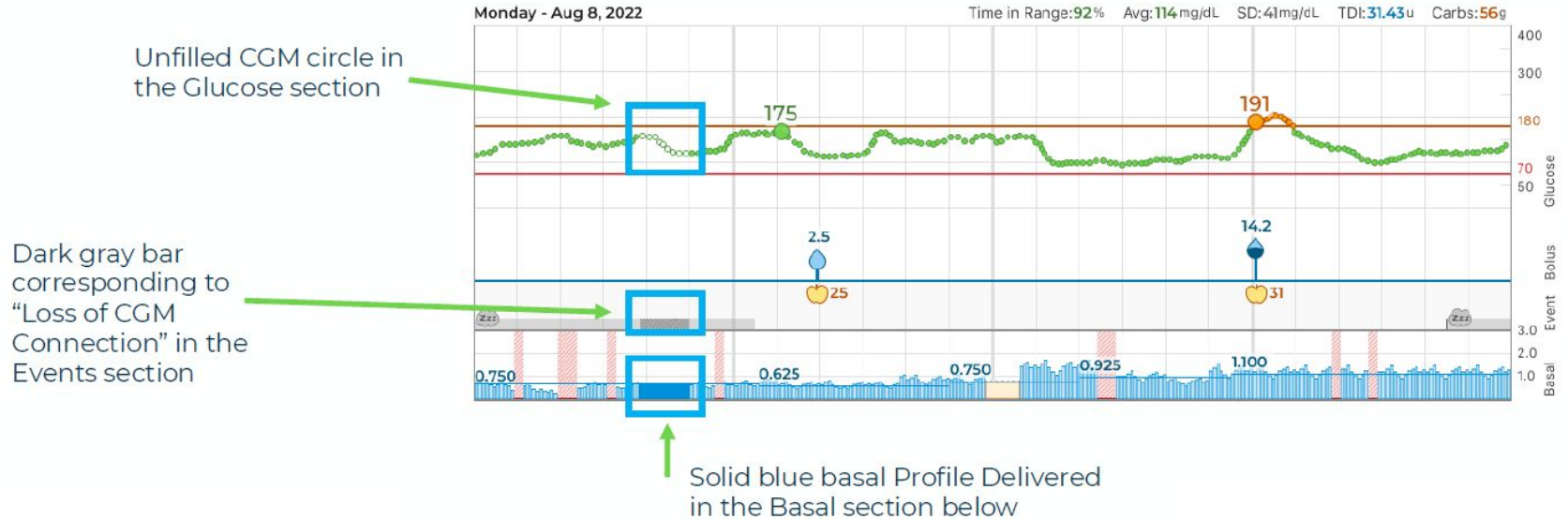 Manual Stop

 Shutdown

**Pump Alarms:** 11:13am - Pump low power; 12:34pm - Button alarm; 1:55pm - Occlusion. **CGM Alerts:** 8:35am - CGM out of range; 11:00pm - Transmitter error; 11:28pm - Transmitter expired.

**CGM Alerts:** 8:35am - CGM out of range; 11:00pm - Transmitter error.

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## Overview Report

- This date range contains one or more incomplete therapy days. Data from the affected day(s) is excluded from daily averages. Affected day(s): Mar 22, Mar 23, Mar 27

## Daily Timeline

- This date range contains one or more incomplete therapy days. This may result in an incomplete graph. Affected day(s): Mar 22, Mar 23, Mar 27

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## Overview Report

- This date range contains one or more days with no data. These day(s) are excluded when calculating daily averages. Affected day(s): Mar 24, Mar 25, Mar 26.

## Daily Timeline

- This date range contains one or more days with no data. Affected day(s): Mar 24, Mar 25, Mar 26.

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## Overview Report

### Important Therapy Information

- This date range contains data from more than one pump. Data from each pump is displayed separately.

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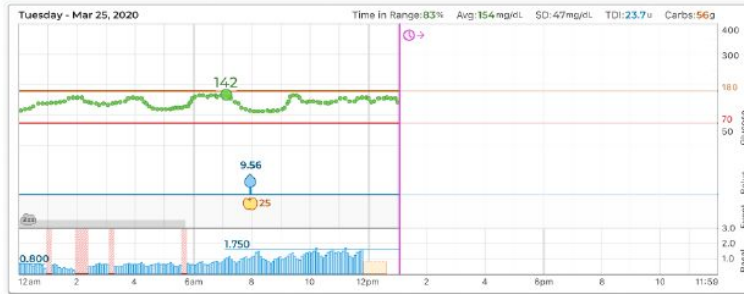
## Overview Report

### Important Therapy Information

- This date range is affected by a pump time change. Data from the affected day(s) is excluded from daily averages.

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→ Pump date change forward occurred from Tuesday, Mar 25 to Wednesday, Mar 26.

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## Overview Report

### ⚠ Important Therapy Information

- A pump date change forward occurred in this date range. This may result in incomplete days or days with no data.

## Daily Timeline

### ⚠ Important Therapy Information

- This date range contains a pump date change from Mar 24 to Mar 27. Graphs with affected days have been separated to before and after the date change. Extended bolus durations may not display accurately.

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## Overview Report & Daily Timeline

### Important Therapy Information

- This date range contains data from pumps with older software versions that is unsupported on Tandem Source. Only data from after the pump software update is displayed and used in calculations. Pump software updated: Aug 8.

**Important Safety Information:** The pump and each system below is indicated for use in individuals 6 years of age and greater. The pump and each system is intended for single user use. Refer to the User Guide for a list of compatible insulins.

The t:slim X2 insulin pump with Basal-IQ technology (the Basal-IQ System) consists of the t:slim X2 insulin pump, which contains Basal-IQ technology, and a compatible continuous glucose monitor (CGM, sold separately). The t:slim X2 insulin pump is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in people requiring insulin. The t:slim X2 insulin pump can be used solely for continuous insulin delivery and as part of the Basal-IQ System. When used with a compatible CGM, the t:slim X2 insulin pump with Basal-IQ technology can be used to suspend insulin delivery based on CGM sensor readings.

The t:slim X2 insulin pump with Control-IQ technology (the Control-IQ System): The t:slim X2 insulin pump is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The pump is able to reliably and securely communicate with compatible, digitally connected devices. Control-IQ technology is intended for use with a compatible continuous glucose monitor (CGM) and the t:slim X2 insulin pump to automatically increase, decrease, and suspend delivery of basal insulin based on CGM readings and predicted glucose values. It can also deliver correction boluses when the glucose value is predicted to exceed a predefined threshold.

**WARNING:** Control-IQ technology should not be used by anyone under the age of 6 years old. It should also not be used in users who require less than 10 units of insulin per day or who weigh less than 25 kilograms.

Do not use the t:slim X2 insulin pump with Control-IQ technology if using hydroxyurea. Neither System is indicated for use in pregnant women, people on dialysis, or critically ill users. Users of the pump and either System must be willing and able to use the insulin pump, CGM, and all other system components in accordance with their respective instructions for use; test blood glucose levels as recommended by their healthcare provider; demonstrate adequate carb-counting skills; maintain sufficient diabetes self-care skills; see healthcare provider(s) regularly; and have adequate vision and/or hearing to recognize all functions of the pump, including alerts. If your CGM readings do not match your symptoms or expectations, use a blood glucose meter to make diabetes treatment decisions. The t:slim X2 pump must be removed before MRI, CT, or diathermy treatment. Visit [tandemdiabetes.com/safetyinfo](https://tandemdiabetes.com/safetyinfo) for additional important safety information.

The information on the Tandem t:slim mobile app display may not be identical to the current status of the user's pump. Wireless uploads from the Tandem t:slim mobile app to the cloud-based Tandem Source platform require a compatible phone and an internet or wireless data connection. Uploads to the Tandem Source platform do not take place in real time and should not be relied upon by healthcare providers or caregivers for remote patient monitoring. Standard carrier data rates may apply.

Tandem t:slim mobile app: The Tandem t:slim mobile app is intended to be a secondary display for compatible Tandem Diabetes Care insulin pumps and is capable of wirelessly uploading pump data it receives into the Tandem Source platform. The Tandem t:slim mobile app is not intended to control an insulin pump and is not intended to be a replacement for the information displayed on your insulin pump. Dosing decisions should not be made based on the secondary display device. The user should follow instructions on the continuous glucose monitoring system and insulin pump. This secondary display device is not intended to replace self-monitoring practices as advised by a physician.

The Tandem Source platform is intended for use by individuals with diabetes mellitus who use Tandem Diabetes Care insulin pumps, their caregivers, and their healthcare providers in home and clinical settings. The Tandem Source platform supports diabetes management through the display and analysis of information uploaded from Tandem insulin pumps.

© 2024 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, Control-IQ, Basal-IQ, Tandem Source, t:slim, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. ML-1014679\_A